

Instruction for use

【The product name】 Surgical Mask

【scope of application】

For clinical medical personnel to wear during operation, covering the user's mouth, nose and jaw, to prevent direct penetration of pathogens, microorganisms, body fluids, particles and other physical barriers.

【specifications and models】

Planar hanging lug: 175mm×95mm.

【structure composition】

Medical surgical mask is composed of surface layer, filter layer, inner layer, nose clip and mask belt. single use only.

【Instructions】

1. Front and back identification: unfold the stack, nose clip side on, blue face outwards.
2. Unroll the mask and push your hands flat against your face.
3. Hang to both ears and press the bridge of the nose from inside out with your fingertips, moving along the shape to the side.
4. Fully extend the mask from top to bottom so that it covers the whole area and fits the face.





【contraindications, precautions, other warnings and reminders】

1. Do not use if you are allergic to non-woven materials.
2. Check the packaging before use. Any damaged packaging is strictly prohibited.
3. This product is disposable. After use, it shall be destroyed according to medical waste.
4. Please keep it in a clean, dry and ventilated place.
5. In case of allergy, please stop using immediately.

【restrictions on use】

1. It shall not be used in the medical environment required by non-protective isolation and the medical environment required by non-special purification.
2. It shall not be worn by clinical personnel during invasive, aseptic or invasive operations.

【Symbol descriptions】

	Avoid the sun
	To avoid the rain
	Disposable use
	Do not use if the package is damaged

【storage conditions】 Store in a ventilated, clean and dry place at room temperature. Do not come into contact with liquid or harmful gas.

【term of validity】 3 years. This validity period is for accelerated aging test.